[User Manual]

Digital Hand - R

  
*Simplicity*  
MASTER

Rev.1.2  
2017.01.20

 **BOMTECH**  
ELECTRONICS CO., LTD.

Bomtech Bldg.,34,Horyeong-ro 49-gil,  
Seocho-gu,Seoul,Korea

Tel : +82-2-523-8295/6 Fax : +82-2-523-8297

E-Mail : [bomtech@bomtech.net](mailto:bomtech@bomtech.net)

Website : [www.bomtech.net](http://www.bomtech.net)  
[www.bomtech.co.kr](http://www.bomtech.co.kr)

## How to verify the product authenticity

1. Search the [hiddetag] via App store, Google play, 360 market or Baidu, [hiddetag] or [hiddetag C.O.P].

2. HiddenTag C.O.P  
Install a HiddenTag C.O.P.  
"hiddetag C.O.P" APP

3. Run the app and click [Verify authenticity now] button.

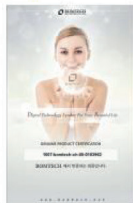
4. Scan the hologram by adjusting the camera back and forth to focus on it.

5. You can check the image of authenticity on your smart phone after scanning.

Authenticity image

A counterfeit image

6. If you touch the lower side of image, it is linked to BOMTECH website.

---

## Composition

---

- Hand - piece

- Adapter

- Hand - piece Stand

- Needle Cartridge (Optional)
- Hygienic Individual Packaging

---

## How to use

---

### 1) Setting

- 1) Place the device and stand on a flat surface.
- 2) Insert the DC adapter to the hand-piece socket and plug the adapter into a power outlet.

### 2) Needle Cartridge Connection & Removal

- MTS Needle Cartridge Connection

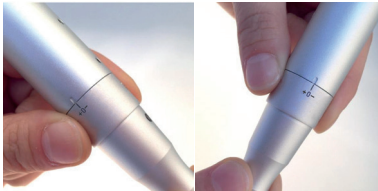
- PMU Needle Cartridge Connection


\*If the cartridge doesn't correctly fit into the hand-piece, It may occur malfunction.

### 3) Needle Length Adjustment

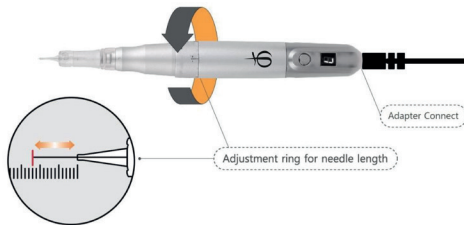Adjust needle length by turning the adjustment ring.

PMU

MTS

#### 4) How to use hand-piece

---

## How to Operate

---

### 1) Power ON/OFF

Press the power button for 2 seconds to turn on/off the power.

### 2) Adjustment of speed level (7 Levels)

- Press the speed level button to change the speed level.
  - Default speed level is 1. Current speed displays on LCD screen.
  - The device memorizes the last speed level you selected.
- You can restart at the last speed level (if the device is powered on).

---

## Precaution before Use

---

**Please read this manual carefully before use.**

- **1) Warning**

- 1) Do not apply excessive pressure when using the device. Be careful not to drop the device.
- 2) For MTS operation, do not use on thin and sensitive skin especially around lips and eyes.
- 3) When rashes, infections or allergies appear on your face, stop using and consult a doctor.
- 4) Check if a container package is unsealed or damaged before use.  
Check the expiry date as well.
- 5) Keep the device on a flat ground and prevent from any impact or vibration.
- 6) Check if whole device works properly before use.
- 7) Needle cartridge is disposable. Do not reuse.
- 8) Once a needle cartridge is damaged, it must be discarded even if it is not used.

- **2) General Precaution**

- **Do not use it on the power above 240V.**  
Otherwise, it might cause fire, electric shock or malfunction
- **Do not repair, disassemble or alter the device.**  
Otherwise, it may cause malfunction.
- **Do not handle the device or detach the power cords with wet hands.**  
Otherwise, it may cause electric shock.
- **Do not scratch, cut or bend the adapter or cord.**  
Otherwise, it may cause fire or electric shock.
- **Unplug the power code when not in use for a long time.**  
Otherwise, it may cause fire.
- **For professionals only.**  
Otherwise, it might cause accidents or malfunction.

- **3) Safety**

- 1) Do not shake or pull with excessive force when using adapter.
- 2) Keep the machine in a dry place.
- 3) The machine must be kept clean.
- 4) Keep the machine at room temperature and a well-ventilated place.
- 5) Keep out of the reach children.
- 6) Read the precaution carefully before use.

- **4) Precaution before Use**

- Please consult a doctor before operating to a man with the symptoms as follows.
  - 1) A man who is regularly treated for high body temperature.
  - 2) A man who suffers from chronic disease which gets worse in a temperature over 38 °C.
  - 3) A man who has taken steroid for long term or is vulnerable to capillarectasia due to liver disease.
  - 4) A man who suffers from melanosis in the face.
  - 5) A man who is unsure about his/her skin condition.
  - 6) A man who suffers from malignant tumor, infectious disease, hemophilia, tuberculosis, epilepsy, leukemia, or allergic disease.
  - 7) A man who suffers from arthritis, high blood pressure, diabetes, keloid, or heart disease.
  - 8) A man who suffers from irregular blood pressure, or a pregnant woman, a breast feeding woman, and a fertile woman.
  - 9) A man who is physically extremely weak.
  - 10) A man who suffers from skin problem on the treatment area [sensitive skin, atopic skin].
  - Use on healthy skin.
  - Avoid using on trouble skins (dermatitis, pimple) or allergic to metal.
  - Keep the treatment area and hands clean.
  - Do not use for newborn infant, toddler, child and animal.

- **5) Interaction**

As such, the Simplicity machine is a multipurpose electric propulsion machine capable of inflicting vertical punctures with adjustable stroke frequency and penetration depth and can be used in a multiple variety of techniques and treatments depending on the cartridge\* in use and machine manipulation technique

- Voltage or noise might cause malfunction. The machine must be kept clean.

---

## Maintenance and inspection

---

### • 1) General maintenance

- Power off when not in use.
- Wipe the surface with a dry cloth.
- Keep the machine at room temperature and in a well ventilated place.
- Keep away from dropping, jarring, or shaking the product.
- Keep away from magnetic forces.
- Remove the power plug completely from the power outlet during a thunder storm.
- Do not repair, disassemble or alter of the device by non-professional staff.
- Keep out of reach of children.

### • 2) Routine inspection

- Check the adopter for any damage before use.
- When you use the device which has not been used for a long time, make sure that the device works properly according to the manual.

---

## Product Warranty

---

The Product Warranty provided by BOMTECH ELECTRONICS CO., LTD. [called "BOMTECH" ] applies only to 'Digital Hand - R' you purchase for your use.

### Warranty Period

- A warranty period of 600 hours for 'Digital Hand-R' means that BOMTECH provides warranty service without charge for during warranty period.
- BOMTECH assures to hold spare parts at least for 2 years after discontinuing production of 'Digital Hand-R'.

### What this Warranty Does not Cover

- Side effect due to a careless treatment.
- Failure or damage resulting from misuse, abuse, accident, modification, unsuitable physical or operating environment, natural disasters, power surges, improper maintenance, or use not in accordance with product information materials.
- Malfunction or damage to product while in transit. Malfunction by using unauthorized components.

### How to Obtain Warranty Service:

- If you have a technical problem, contact to the place of purchase.
  - Please ship your product for a warranty service to the place of purchase.
- Shipper shall bear the shipping costs to send/receive the product.

Digital Hand-R	Serial No.	Place of Purchase	Date of Purchase

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