[User Manual]

Digital Pen - A

Simplicity
ARTIST

Rev.1.2
2017.01.20

 **BOMTECH**
ELECTRONICS CO., LTD.

Bomtech Bldg., 34, Hyoryeong-ro 49-gil,
Secho-gu, Seoul, Korea

Tel : +82-2-523-8295/6 Fax : +82-2-523-8297

E-Mail : bomtech@bomtech.net

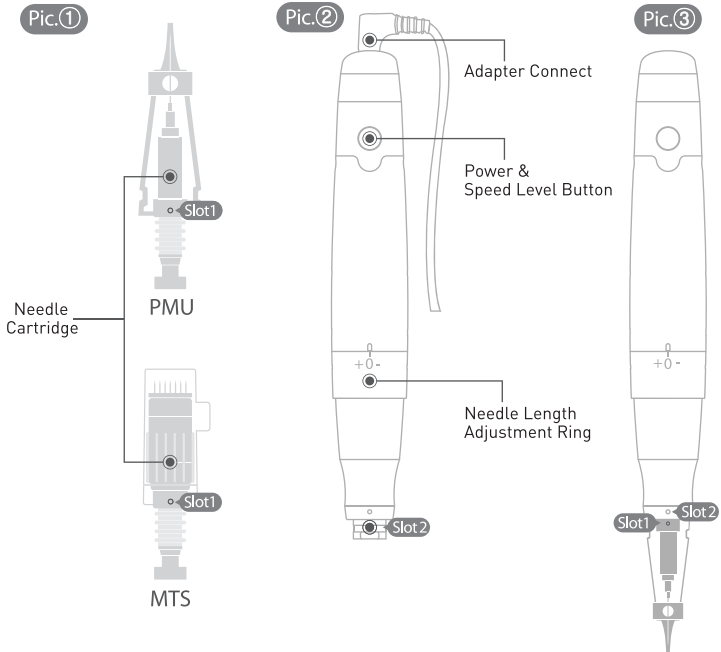
Website : www.bomtech.net
www.bomtech.co.kr

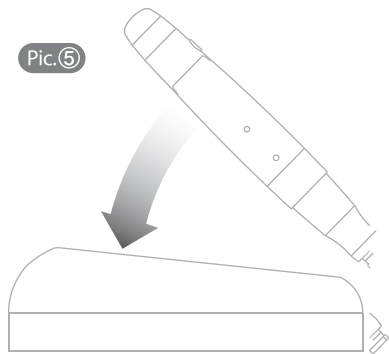
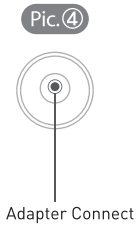
Composition


1. Hand-piece 2. Adapter (5V-1A) 3. Hand-piece Holder 4. Needle Cartridge (Optional)


Parts Name

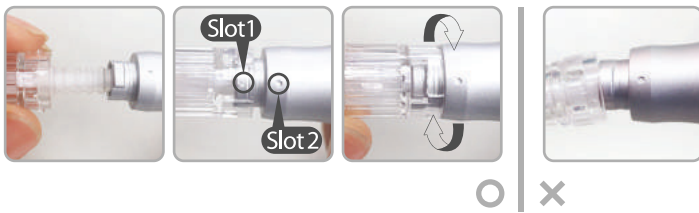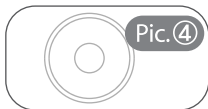*If the cartridge doesn't correctly fit into the hand-piece, it might malfunction.

1. Setting

- 1) Connect the adapter to the adapter connect **Pic. 4**
- 2) Connect the needle cartridge to the hand-piece **Pic. 3**

2. How to Use Needle Cartridge

• Needle Cartridge Connection

- 1) Take a needle cartridge **Pic. 1** out of the sterilized package.
- 2) Fitting round slots 1 into 2 of the hand-piece **Pic. 2** and push the needle cartridge horizontally up to the end **Pic. 3**
- 3) Lock the cartridge by turning it to the counter clockwise.
- 4) Adjust the needle length by turning the adjustment ring.
- 5) Operate PMU/MTS treatment by using the hand-piece **Pic. 3**.
- 6) To remove the needle cartridge, turn it counter-clockwise and detach it from the hand-piece.

1. How to Use Hand-piece

- 1) Grasp the hand-piece straightly or obliquely for treatment.
- 2) If you put down the hand-piece on the stand during operation, it automatically stops.
- 3) Disconnect hand-piece and adapter when finishing using.

2. Adjustment of Speed Level and Needle Length

1) Power ON/OFF and adjustment of speed level

(1) Power (ON/OFF)

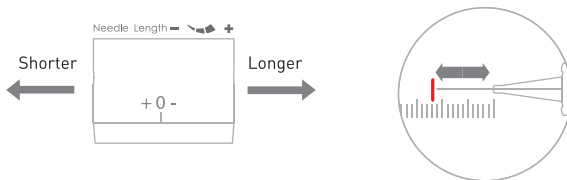
- Press the power button (Pic.②) for 2 seconds to turn on/off the power.

(2) Adjustment of speed level

- Press the speed level button to change the speed level (5 Levels).
- The device memorizes the last speed level of operation and it restarts from the last speed level if the power keeps turning on.

3) Needle Length Adjustment

Adjust the needle length by turning the adjustment ring.

Precaution before Use

Please read this manual carefully before use.

- **Warning**

- 1) Do not apply excessive pressure when using the device.
- 2) Do not use on thin and sensitive skin especially around lips and eyes.
- 3) When rashes, infections or allergies appear on your face, stop using and consult a doctor.
- 4) Check if a container package is unsealed or damaged before use.
Check the expiry date as well.
- 5) Keep the device on a flat ground to prevent from any impact or vibration.
- 6) Check if whole device works properly before use.
- 7) Needle cartridge is disposable. Do not reuse.
- 8) Once a needle cartridge is damaged, it must be discarded even if it is not used.

- **Interaction**

- Do not use with other machines.

- **General Precaution**

- **Do not use it on the power above 240V.**
Otherwise, it might cause fire, electric shock or malfunction
- **Do not repair, disassemble or alter the device.**
Otherwise, it may cause malfunction.
- **Do not handle the device or detach the power cords with wet hands.**
Otherwise, it may cause electric shock.
- **Do not scratch, cut or bend the adapter or cord.**
Otherwise, it may cause fire or electric shock.
- **Unplug the power code when not in use for a long time.**
Otherwise, it may cause fire.
- **For professionals only.**
Otherwise, it might cause accidents or malfunction.

- **Precaution before Use**

- Please consult a doctor before operating to a man with the symptoms as follows.
 - 1) A man who is regularly treated for high body temperature.
 - 2) A man who suffers from chronic disease which gets worse in a temperature over 38°C.
 - 3) A man who has taken steroid for long term or is vulnerable to capillarectasia due to liver disease.
 - 4) A man who suffers from melanosis in the face.
 - 5) A man who is unsure about his/her skin condition.
 - 6) A man who suffers from malignant tumor, infectious disease, hemophilia, tuberculosis, epilepsy, leukemia, or allergic disease.
 - 7) A man who suffers from arthritis, high blood pressure, diabetes, keloid, or heart disease.
 - 8) A man who suffers from irregular blood pressure, or a pregnant woman, a breast feeding woman, and a fertile woman.
 - 9) A man who is physically extremely weak.
 - 10) A man who suffers from skin problem on the treatment area (sensitive skin, atopic skin).
- **Use on healthy skin.**
 - Avoid using on trouble skins (dermatitis, pimple) or allergic to metal.
- **Keep the treatment area and hands clean.**
- **Do not use for newborn infant, toddler, child and animal.**

- **Safety and Usage Information**

- 1) Do not shake or pull with excessive force when using adapter.
- 2) Keep the machine in a dry place.
- 3) The machine must be kept clean.
- 4) Keep the machine at room temperature and a well-ventilated place.
- 5) Keep out of the reach children.
- 6) Read the precaution carefully before use.

Warranty

- 1) If you have a technical problem, contact to the place of purchase.
- 2) BOMTECH provides the after-sales service within 14 days from the receipt of an application form.
- 3) BOMTECH assures to hold spare parts at least for 2 years after discontinuing production of 'Digital Pen-A'
- 4) A warranty period of 600 hours for 'Digital Pen-A' means that BOMTECH provides warranty service without charge for during warranty period.
- 5) The warranty excludes coverage for the following;
 - Damage resulting from mishandling the device, careless treatment, unauthorized repair, or unauthorized modification.
 - Malfunction or damage to product while in transit.
 - Malfunction by using unauthorized components.
 - Troubles due to natural disasters (fire, slat or water, etc.)

Maintenance

1. General Maintenance

- Power off when not in use.
- Wipe the surface with a dry cloth.
- Keep the machine at room temperature and in a well ventilated place.
- Keep away from dropping, jarring, or shaking the product.
- Keep away from magnetic forces.
- Remove the power plug completely from the power outlet during a thunder storm.
- Do not repair, disassemble or alter of the device by non-professional staff.

2. Routine Inspection

- Check the adopter for any damage before use.
- When you use the device which has not been used for a long time, make sure that the device works properly according to the manual.

Product Warranty

The Product Warranty provided by BOMTECH ELECTRONICS CO., LTD. (called "BOMTECH") applies only to 'Digital Pen - A' you purchase for your use.

Warranty Period

- A warranty period of 600 hours for 'Digital Pen-A' means that BOMTECH provides warranty service without charge for during warranty period.
- BOMTECH assures to hold spare parts at least for 2 years after discontinuing production of 'Digital Pen-A'.

What this Warranty Does not Cover

- Side effect due to a careless treatment.
- Failure or damage resulting from misuse, abuse, accident, modification, unsuitable physical or operating environment, natural disasters, power surges, improper maintenance, or use not in accordance with product information materials.
- Malfunction or damage to product while in transit. Malfunction by using unauthorized components.

How to Obtain Warranty Service:

- If you have a technical problem, contact to the place of purchase.
 - Please ship your product for a warranty service to the place of purchase.
- Shipper shall bear the shipping costs to send/receive the product.

Digital Pen-A	Serial No.	Place of Purchase	Date of Purchase

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